

INSTRUCTIONS FOR TRANSPORT BOXES

Ready for despatch in only 6 steps

1 Transport box

Please use blue transport box for storage and transport of medical devices used on patients. Enclosures/contents: Hand-over note, plastic bag for packing products, including seals to close transport box and a plastic bag for packing transport box.

2 Preparation of medical devices

Please clean the medical devices according to the instructions for preparation in hospital.

3 Packaging and labelling

Pack each pre-cleaned medical device individually in the provided plastic bag, together with respective manufacturer's original data/original label*. Check the transport box for complete-ness:

- Medical devices packaging
- ☑ Hand-over note
- Manufacturer's original data/original label*

*PLEASE NOTE

Manufacturer's original data/original label applies only to initial submission of medical devices.

4 Completing the hand-over note

Carefully fill in the hand-over note, sign it and place it in the transport box.



Sealing the transport box

Close the lid of the transport box and push each supplied seal into openings provided on the left and right front of the closed box.

6 Despatch

Pack blue transport box in supplied plastic bag and close it. The transport box is now ready for collection.

Free Service-Hotline: 00800 82648273